
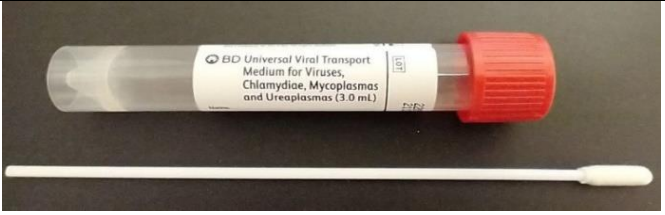


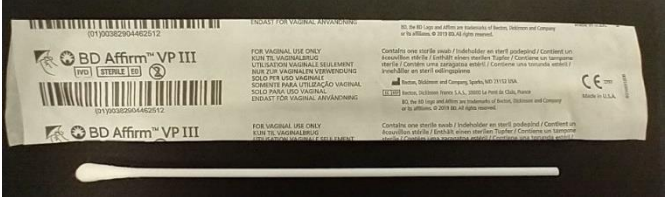
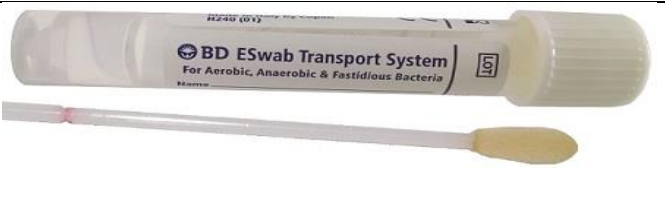


SWAB REFERENCE GUIDE

| Specimen Type | General Tests Associated | Photo |
|---|--|---|
| <p>Nasopharyngeal (NP swab) With UTM</p> | <ul style="list-style-type: none"> • Respiratory Pathogen • Influenza A/B (PCR) • RSV (PCR) • SARS-CoV-2 Antigen • SARS-CoV-2 RNA by RT-PCR |  |
| <p>Nasal Swab with UTM</p> | <ul style="list-style-type: none"> • SARS-CoV-2 Antigen • Influenza A/B (PCR) • RSV (PCR) • SARS-CoV-2 RNA by RT-PCR |  |
| <p>Nasal Swab (Dry) Covid Swabs must be tested within 1 hour Flu and RSV swabs must be tested within 4 hours</p> | <ul style="list-style-type: none"> • SARS-COVID19 RNA Rapid • Influenza A/B - Rapid • RSV- Rapid |  |
| <p>Red Duo Swab</p> | <ul style="list-style-type: none"> • Rapid Strep A |  |
| <p>Affirm Swab</p> | <ul style="list-style-type: none"> • Affirm DNA Panel |  |
| <p>E Swab</p> | <ul style="list-style-type: none"> • Cultures, aerobic and anaerobic • MRSA • Group B Strep |  |