

Blanchard Valley Health System
 Laboratory Point of Care Testing
 DCA Vantage HbA1c Quarterly Maintenance Log

Site:

SN:

Year:

Ran Optical Test Cartridge - record results

Quarter	Date	Trained Operator Initials	Removed and Cleaned the Cartridge Spring and Cartridge Area	Checked the Air Filter	Replaced the Air Filter (if needed)	Mean Transmittance	Standard Deviation	Drift T	Drift Max	Compared values with recommended readings and acceptable	Remedial Action	Supervisory Review Date & Initials
						(Mean T) 0.95-1.05	(SD T) <0.0015	No applicable limits	<0.0280	Yes/No	(If needed)	
Jan												
Feb												
Mar												
Apr												
May												
Jun												
July												
Aug												
Sept												
Oct												
Nov												
Dec												

Supervisory review by: _____

Date: _____