



ID NOW™ Cleaning Log

Form revised 8/9/2023 LTR

The ID NOW™ is maintenance-free and has no serviceable parts. In the case of instrument failure or damage, contact Abbott Technical Support at 855-731-2288 or ts.scr@abbott.com

The ID NOW™ can be cleaned using 70% ethanol, 70% isopropanol, or 10% bleach solution, on a damp, lint free cloth. 70% ethanol and isopropanol wipes are acceptable for use on the ID NOW™. Do not spray or pour solution directly onto instrument when cleaning. Ensure no excess liquid is used when cleaning as it may damage the instrument.

Abbott recommends that the exterior instrument surfaces and the surfaces visible under the open lid be cleaned daily. Clean the surrounding bench area. Clean instrument and surrounding areas immediately after possible patient sample contamination.

- Do not disassemble the instrument for cleaning
- Do not immerse in water or cleaning solutions
- Do not clean with soap or other solutions

INSTRUMENT SERIAL NUMBER _____ MONTH _____ YEAR _____

DAY	PERFORMED BY: (INITIALS)	COMMENTS/CORRECTIVE ACTION	DAY	PERFORMED BY: (INITIALS)	COMMENTS/CORRECTIVE ACTION
1			17		
2			18		
3			19		
4			20		
5			21		
6			22		
7			23		
8			24		
9			25		
10			26		
11			27		
12			28		
13			29		
14			30		
15			31		
16					

FOR EXTERNAL USE. PRINT AND DISTRIBUTION PERMITTED.

© 2022 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. 120004482 Rev.4 07/21

Monthly Supervisory Review by: _____ Date: _____