


STATSTRIP Xpress[®] 2

Glucose Monitoring System Quick Operating Guide for Blood Sample



Left Button
Mode Button
Right Button

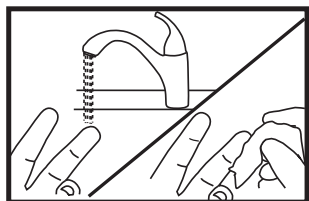


1 The Home screen


2 Wash patient's hand thoroughly and massage finger to stimulate blood flow.

3 Insert a test strip into the meter. A test strip with a blood drop will display.

4 Use a safety lancet to puncture the finger.

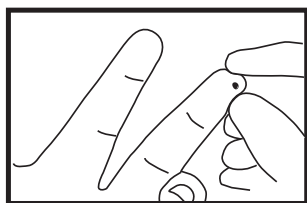

5 Squeeze the finger to form a drop of blood. Wipe away the first drop of blood then squeeze the finger again to form a second drop of blood.

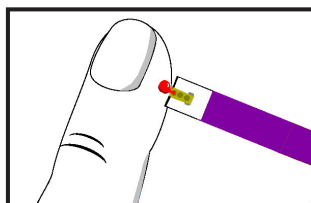

6 When the blood drop appears, touch the end of the test strip to the blood drop until the test strip fills and the meter beeps.

7 **Warning!**

The test strip must fill completely upon touching the blood droplet. If the test strip does not fill completely, do not touch the blood droplet a second time. Discard the test strip and repeat the test with a new test strip.



8 Glucose test results are available on-screen in 6 seconds. There is one long beep when the results are ready. There are 3 short beeps if test results are outside the range of the test strip.

9 Remove the test strip and discard into an appropriate biohazard container.

10 When patient testing is complete, the StatStrip Xpress2 Glucose Hospital Meter should be cleaned and disinfected after use prior to testing with a new patient. For instructions, see Cleaning and Disinfecting the Meter section in the Instructions for Use Manual.

11 Test results are automatically saved. If no activity for 1 minute, the meter will time-out: screen goes blank.

nova
biomedical

Nova Biomedical, 200 Prospect Street, Waltham, MA 02454
Tel: 800-545-6682 • www.novabiomedical.com