
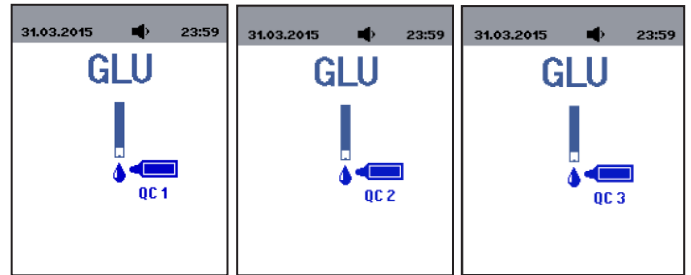


STATSTRIP Xpress[®] 2


Glucose Monitoring System Quick Operating Guide for QC Controls

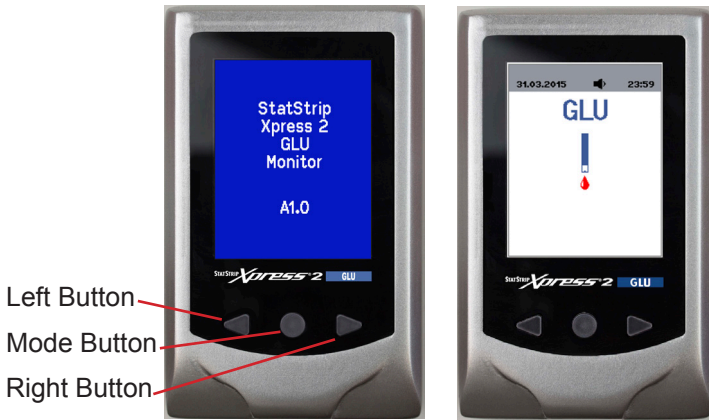

1 The Home screen

2 Insert a test strip into the meter. A test strip with a blood drop will display.

3 Identify the sample as a Control; use the Left or Right button to find the desired control level: QC1, QC2, or QC3. Gently mix the StatStrip Glucose Control Solution before each use.

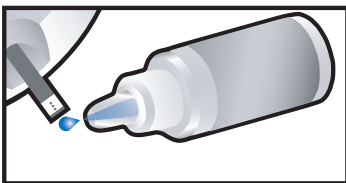

4 Discard the first drop of control solution from the bottle to avoid contamination. Touch the end of the test strip to a drop of control solution until the test strip fills and the meter beeps.

5 **Warning!**

The test strip must fill completely upon touching the QC droplet. **Do not add a second QC drop to the test strip.** Discard the test strip and repeat the test with a new test strip.

Note!

A quick beep sounds when sufficient control solution has been added to the test strip.



6 Recap the control solution. Glucose quality control results are available on-screen in 6 seconds. There is one long beep when the results are ready. There are 3 short beeps if test results are outside the range of the test strip.

7 Remove the test strip and discard into an appropriate container.

8 Test results are automatically saved. If no activity for 1 minute, the meter will time-out: screen goes blank.

nova[®]
biomedical

Nova Biomedical, 200 Prospect Street, Waltham, MA 02454
Tel: 800-545-6682 • www.novabiomedical.com