

Blanchard Valley Health System: Point of Care Testing Abbott Afinion2 - Maintenance Log

Year: _____

Serial Number: _____

Monthly:

1. **Clean the outside of the analyzer and the touch display** with a clean, lint-free and non-abrasive cloth dampened in water or mild detergent.
2. **Clean the cartridge chamber** using a cleaning swab wet with 3 drops of water.
 - a. Wipe off any residual liquid from the chamber using a new, dry cleaning swab.
 - b. Touch the orange Patient or blue QC icon to open the chamber.

	JAN	FEB	MAR	APR	MAY	JUN	JUL	AUG	SEP	OCT	NOV	DEC
DATE												
INITIALS												

CORRECTIVE ACTION:

REVIEWED BY: _____ **DATE:** _____